# A-02 Headphones

AUDEARA

|                                      |     |
|--------------------------------------|-----|
| <b>ENGLISH</b>                       |     |
| A-02 HEADPHONES GUIDE.....           | 5   |
| <b>DEUTSCH</b>                       |     |
| A-02 KOPFHÖRER – ANLEITUNG .....     | 21  |
| <b>ESPAÑOL</b>                       |     |
| GUÍA DE AURICULARES A-02 .....       | 37  |
| <b>FRANÇAIS</b>                      |     |
| GUIDE DES ÉCOUTEURS A-02 .....       | 53  |
| <b>ITALIANO</b>                      |     |
| A-02 GUIDA ALLE CUFFIE.....          | 69  |
| <b>MAGYAR</b>                        |     |
| A-02 FEJHALLGATÓ ÚTMUTATÓ .....      | 85  |
| <b>NEDERLANDS</b>                    |     |
| A-02 HOOFDTELEFOONGIDS .....         | 101 |
| <b>POLSKI</b>                        |     |
| A-02 PRZEWODNIK PO SŁUCHAWKACH ..... | 117 |
| <b>PORTUGUÊS</b>                     |     |
| GUIA DE FONES DE OUVIDO A-02 .....   | 133 |

# Contents

## OVERVIEW

|                     |   |
|---------------------|---|
| What's in the box   | 6 |
| Headphones overview | 8 |

## HEADPHONES

|                         |    |
|-------------------------|----|
| Connect your Headphones | 11 |
| Tailor your Headphones  | 15 |
| Headphones tips         | 16 |

# What's in the box


Headphones


Carry Case


## HEADPHONES ACCESSORIES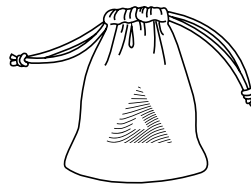


Accessory Bag

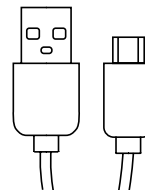USB-C Power Cable

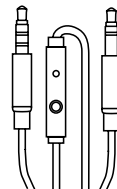
3.5mm Audio Cable  
(with microphone)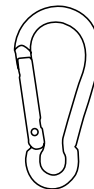


Carabiner

# Headphones


Left Ear Cup  
Headphone Controls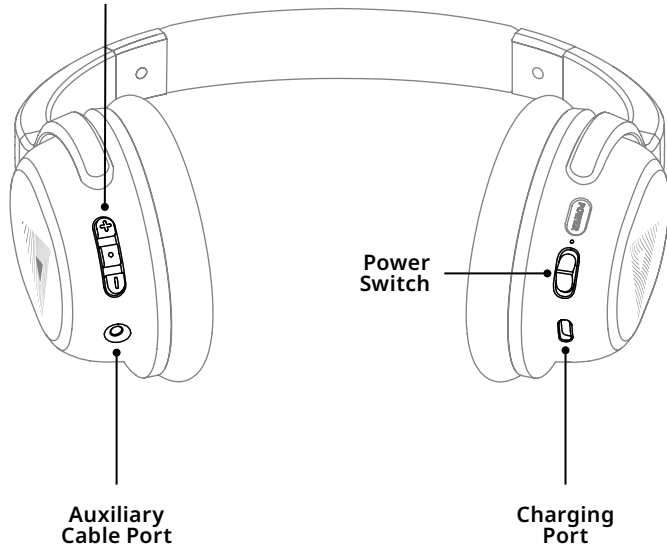

LEFT EAR CUP

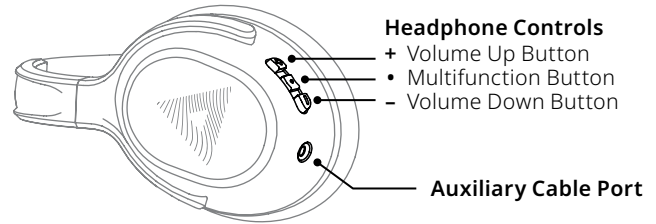RIGHT EAR CUP
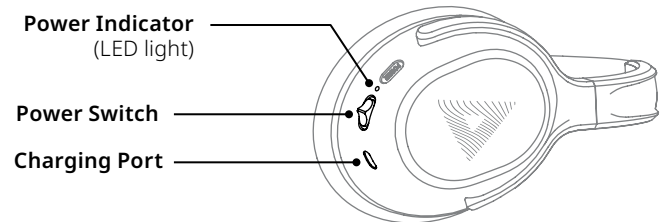
# Connect your Headphones

Congratulations for putting your hearing health first by choosing Audeara. We're excited to help you feel connected and rediscover the joy of listening.

Your Audeara Headphones can connect to any Bluetooth® enabled device. This allows you to enjoy music and make audio and video calls wirelessly.


## 1. Place your Headphones in Pairing Mode

Power your Headphones on by flicking the Power Switch up

- ▶ This will make the Power Indicator flash blue.
- ▶ Your Headphones will also provide an audible prompt that says “Power On”.

Hold down the Multifunction (middle) Button for three seconds

- ▶ This will make the Power Indicator flash blue and red.
- ▶ When in pairing mode, your Headphones will also provide an audible prompt that says “Pairing” followed by a repeated chime while in pairing mode.

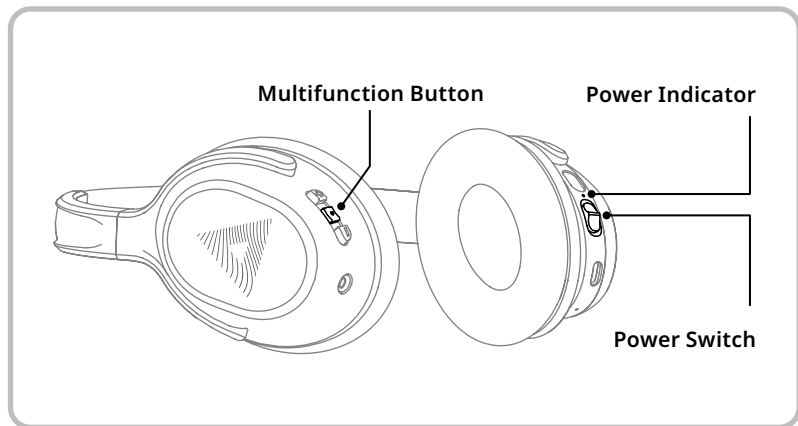## 2. Connect to a mobile device

On your mobile device, navigate to Settings > Bluetooth and ensure Bluetooth is turned on

- ▶ You should see “Audeara A-02” in the device list. If it doesn’t appear, see step 1.

Tap on “Audeara A-02” to pair to your device

- ▶ Once the device is connected, your Headphones will stop flashing blue and red and you will also hear an audible prompt that says “Connected”.

## 3. Connect to a second mobile device (optional)

Your Headphones can pair to two devices at the same time

- ▶ With your Headphones already connected to the first device, repeat steps 1 and 2 with the second device.
- ▶ Turn your Headphones off and then on again using the Power Switch to connect to both paired devices.

# Experience sound tailored to you

Now that you've connected your Headphones to your mobile device, you can tailor your Headphones to your hearing profile with the Audeara app.

We all hear differently - even your left ear hears differently to your right. Connect your Audeara Headphones to our app and take a simple hearing check in just minutes.

Use your results to tailor your Audeara Headphones to your individual hearing profile.


Once tailored, your Headphones store your hearing profile regardless of what device you connect to. You can change your profile at any time using the Audeara app.

Scan to  
Download

# Headphone tips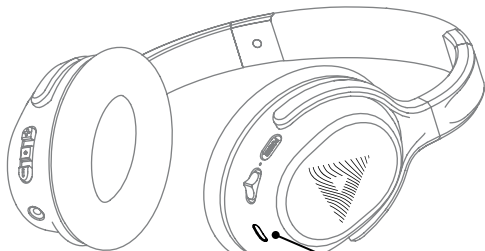

USB-C Power Cable  
USB-C end  
(smaller tip)

Charging Port

## POWER INDICATOR

**Red LED** indicates that the Headphones are charging.

**Blue LED** indicates that the Headphones are charged.

## Charging your Headphones

Your Headphones have a battery life of 30+ hours

- ▶ Use the USB-C Power Cable provided to charge your Headphones.
- ▶ Insert the USB-C end (smaller tip) into the port on the right earcup of the Headphones.
- ▶ Insert the USB-A (bigger tip) into a USB power source. At this stage you will see the Power Indicator light up on the Headphones.

## Multifunction Button

Answer or end a phone or video call

- ▶ If you receive a phone call to a connected mobile device while wearing your Headphones, press the Multifunction Button to answer or hang up.

Pause or play your music

- ▶ If listening to music, press the Multifunction Button to pause or play your music.

## Carry case

Travel with your Headphones

- ▶ Clip the provided Carabiner to the carry case for easy travel.

## FCC AND IC STATEMENT

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

1. This device may not cause harmful interference,
2. This device must accept any interference received, including interference that may cause undesired operation.

CAN ICES-3(B)/NMB-3(B)

## FEDERAL COMMUNICATION COMMISSION INTERFERENCE STATEMENT

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

## FCC CAUTION

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

## RoHS COMPLIANCE

This product is in compliance with the European Union's Restriction of Use of Hazardous Substances in Electrical and Electronic Equipment (RoHS) Directive 2011/65/EU.

## IC COMPLIANCE STATEMENT

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions:

1. This device may not cause interference.
2. This device must accept any interference, including interference that may cause undesired operation of the device.

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CAN ICES-3(B)/NMB-3(B)

## WEEE NOTICE

This symbol on the product indicates that the product must not be disposed of with normal household waste. Instead, it is your responsibility to dispose of your waste equipment by arranging to return it to a designated collection point for the recycling of waste electrical and electronic equipment. By separating and recycling your waste equipment at the time of disposal you will help to conserve natural resources and ensure that the equipment is recycled in a manner that protects human health and the environment.

Audeara offers our customers the opportunity to return products at the end of their functional lives to Audeara for recycling, qualifying products are marked with the WEEE symbol shown above. Delivery cost of any authorised product returned to Audeara will be at the expense of the sender.

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